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VERSION 02

SIDLY

SIDLY Sp. z o.o.
Chmielna 2/31
00-020 Warszawa
Poland

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Authors: SiDLY Sp. z o.o.

Graphic design: SiDLY Sp. z o.o.

Operating instructions recommended for distribution among the customers of the device manufacturer who bought the device model covered by this manual.

The content complies with the requirements of national and EU legislation in force in November of 2015.

Instructions apply effective law on copyright and related rights.

Any changes made to the manual without the knowledge and permission of the author's exempt him from responsibility of disputes resulting from these changes.

All rights to reprint or copy are reserved - only allowed with the permission of the manufacturer.

Any changes made to the manual without the knowledge and permission of the SiDLY's exempt him from responsibility of disputes resulting from these changes

Instruction is available at www.sidly-care.eu

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SiDLY One

SiDLY Sp. z o.o.
Chmielna 2/31
00-020 Warsaw Poland
www.sidly.eu
www.sidly-care.eu

CONTENTS

<u>1..... GENERAL</u>	<u>28</u>
<u>1.1. INTRODUCTION</u>	<u>28</u>
<u>1.2. THE DEVICE IDENTIFICATION</u>	<u>29</u>
<u>2..... SAFETY OF USE AND WARNINGS</u>	<u>33</u>
<u>2.1. GENERAL SAFETY RULES</u>	<u>33</u>
<u>2.2. SAFETY USAGE RULES</u>	<u>34</u>
<u>2.3. USING THE CONNECTORS, PORTS AND BUTTONS</u>	<u>35</u>
<u>2.4. WORKING TEMPERATURE</u>	<u>35</u>
<u>2.5. SAFETY SIGNS</u>	<u>35</u>
<u>2.6. RESIDUAL RISK DESCRIPTION AND ASSESSMENT</u>	<u>35</u>
<u>3..... SIDLY CARE DESCRIPTION</u>	<u>36</u>
<u>3.1. INTENDED USE</u>	<u>36</u>
<u>3.2. TECHNICAL CHARACTERISTICS AND CONSTRUCTION</u>	<u>37</u>
<u>4..... PRELIMINARY OPERATIONS AND PUTTING INTO SERVICE</u>	<u>38</u>
<u>4.1. DEVICE DELIVERY</u>	<u>38</u>
<u>5..... USE OF DEVICE</u>	<u>39</u>
<u>5.1. GENERAL PRINCIPLES</u>	<u>39</u>
<u>5.2. INITIAL START</u>	<u>39</u>
<u>5.3. SWITCHING ON AND OFF</u>	<u>40</u>
<u>5.4. DEVICE SYNCHRONIZATION</u>	<u>41</u>
<u>5.5. THE DEVICE FUNCTIONALITY</u>	<u>43</u>
<u>5.5.1. BAND REMOVAL SENSOR</u>	<u>44</u>
<u>5.5.2. THE MEASUREMENT OF VITAL PARAMETERS</u>	<u>44</u>
<u>5.5.3. DETERMINING A USER'S GEOLOCATION</u>	<u>46</u>
<u>5.5.4. MEDICINES REMINDER</u>	<u>46</u>
<u>5.5.5. SOS BUTTON / PANIC BUTTON</u>	<u>47</u>
<u>5.5.6. FALL DETECTOR</u>	<u>48</u>
<u>5.5.7. GPRS COMMUNICATION</u>	<u>48</u>
<u>5.5.8. PEDOMETER</u>	<u>49</u>
<u>5.5.9. BAROMETER</u>	<u>49</u>
<u>5.6. BATTERY CHARGING</u>	<u>49</u>
<u>6..... MAINTENANCE</u>	<u>51</u>
<u>6.1. CLEANING</u>	<u>51</u>
<u>6.2. STORAGE</u>	<u>51</u>
<u>7..... WITHDRAWAL</u>	<u>52</u>
<u>8..... TESTS AND CERTIFICATES</u>	<u>52</u>
<u>9..... WARRANTY</u>	<u>53</u>

1. GENERAL

Thank You for Your trust and purchase of SiDLY. We hope that thanks to it You provide a feeling of security for yourself and your closes, irrespective of the distance.

1.1. Introduction

WARNING!

This operating instructions should be attached to SiDLY One device.

This operating instructions is a basic equipment of the device. Be familiar with its contents prior to device use. It is recommended to keep the operating instructions for future use.

Please read all of these instructions before using the device for a first time. Reading this entire manual and proceeding carefully with the principles pre-

sent here ensures the safe usage and minimizes the risk of accidents or malfunctions.

These instructions are valid only for the device, which has been delivered with. It is identified with the device according to its type and serial number inscribed on the title page.

WARNING!

Before starting work, make sure you read this entire manual. If any information given in these instructions are incomprehensible please contact for help directly to the manufacturer (business address and contact information are located on the title page of this manual and on the product label).

The SiDLY One device has been designed and manufactured in accordance with safety requirements of Council Directive 93/42/EEG of 14th June 1993 concerning medical devices, but inappropriately utilized may cause hazards to the user health and life or third parties. Therefore, utilizing the device you must follow the provisions of the manual. Only persons who are familiar with the device and its operating instructions can use it.

The SiDLY product must be used only for its intended purpose. The unauthorized usage against the principles set out in this manual will void your warranty. In the event of damage in this respect, the

manufacturer may refuse to replace the device to a new one.

The manufacturer provides a complete SiDLY One device with a dedicated charger and operation manual.

Making any modifications to the device release the manufacturer from the consequences of their implementation and may void Your warranty. Details of the warranty and service are stated in the warranty card.

In case of doubt regarding the operation of the device, observed malfunctions or defects, please contact the manufacturer at the address:

SiDLY Sp. z o.o.
2/31 St. Chmielna
00-020 Warsaw, PL
tel.: + 48 22 379 75 55
e-mail: office@sidly.eu

WARNING!

Any other way of the product use, not described in this operating instructions you must first consult the device manufacturer.

WARNING!

Information important for the user is indicated in such a way!

**NOTE!**

This sign warns of the potential hazard in relation to the device operator or bystanders.

Not following the recommendation marked by this sign can cause accidents. Strictly follow the recommendations specified by this sign.

WARNING!

1. *The manufacturer reserves the right to make changes to the device and its software in order to improve it. These changes will also be introduced on an ongoing basis to the instructions. This does not mean, however, that will be introduced also for devices previously provided to the users. The latest version of the instructions for the device is available on the website www.sidly-care.eu*
2. *The manufacturer is not responsible for accidents resulting from not following the safety usage rules published in this manual and the wider use of the device.*
3. *The device is not serviceable. In the case of any defect manufacturer exchange the device to a new one.*

1.2. The device identification

Identification data of the device are provided on the label, which is located on the back of SiDLY One.

On the label is also:

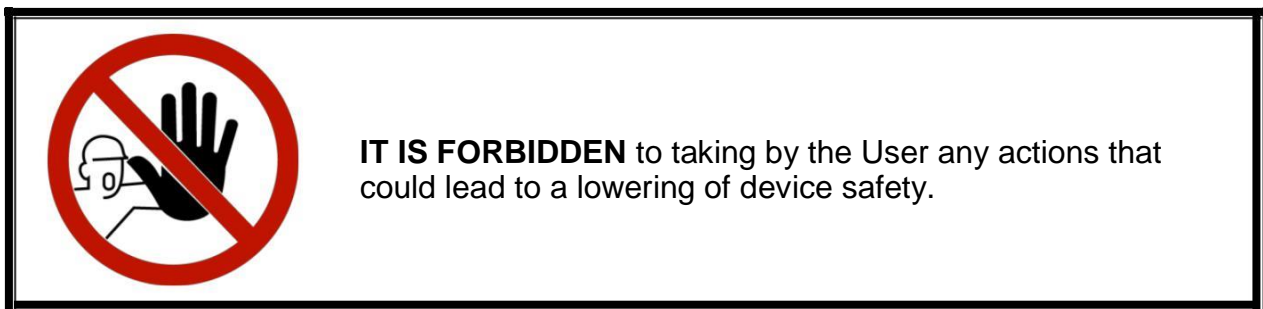
- name and address of the manufacturer,
- serial number,
- sim number
- year of manufacture,
- CE marking

2. SAFETY OF USE AND WARNINGS

2.1. General safety rules

When using the device You are required to respect the general health and safety regulations and the rules laid down for medical devices and consumer electronics devices. In addition to the above You are required to respect the following guidelines, in order to eliminate possible occurrence of hazards:

- do not make any modifications in the device construction,
 - charge built-in battery only the supplied charger that is included as a part of the device kit.
- the device should be used only in accordance with the intended purpose,

Use the SiDLY One device only for the intended purposes and in a manner described by the manufacturer.

Before use, please read these instructions and observe the following rules regarding the elimination of hazards and taking appropriate precautions.

- Use the device only in the intended manner (see ch. 3.1).
 - It is forbidden to use the device by children under 3 years old.
- It is forbidden to use SiDLY Care for purposes other than those specified in this manual.
 - Do not make unauthorized changes to the SiDLY Care construction.
 - Charge the device only a charger supplied with SiDLY One.

2.2. Safety usage rules

**NOTE!**

For the device to work properly it must be 2G GSM range. We recommend You to place precisely the device on his left wrist, correctly and in a way that not create a discomfort feeling.

WARNING!

Do not wear the device for bathing or swimming in the pool.

WARNING!

*Direct strong impact in the band will damage it.
A puncture of the battery by a sharp tool may cause fire due to the release of energy stored in the lithium-polymer battery.*

WARNING!

Remove the device before you will be subjected to MRI (magnetic resonance imaging) or X-rays.

- It is forbidden to use devices that have become wet or frosted.
- It is unacceptable to use the device after You find any damages or improper operation. You should contact with the manufacturer at first.
- It is unacceptable to charge the battery when the device is placed on sb's wrist. Before starting the charging process, remove the device from his hand.
- Nano-SIM card is placed in the device by the manufacturer.
- The device is not intended to perform its functions on the plane because of the lack of possibility of sending SMS messages to the server.
- The device is sensitive to strong impact therefore it is unacceptable to intentionally throwing band.
- We recommend removing the device before you will be subjected to MRI (magnetic resonance imaging – the risk of band damaging) or X-rays (the risk of electronic damaging).

2.3. Using the connectors, ports and buttons

Never use too much force when pressing a connector to USB port, or use excessive force to press the button with the switch.

Such action may cause damage that is not covered by the warranty.

If there is not easy to connect plug and USB port, they probably do not match.

Check whether there is something between the USB and USB socket that can block the correct connection.

Make sure that the plug fits into the socket and that the plug is properly positioned in relation to the socket.

2.4. Working temperature

The device has been designed for work and storage in normal conditions at ambient temperature $-5^{\circ}\text{C} - +45^{\circ}\text{C}$.

If SiDLY One is used or stored at temperatures outside this range it may be damaged.

Do not expose the device to sudden temperature or humidity changes.

During use or charging the battery, SiDLY One may become hot, which is a normal phenomenon.

Do not leave the device for a long time in heated or cooled-site (eg. in a car or under direct sunlight).

2.5. Safety signs

Pictogram (safety sign) on the device informs about an unacceptable method of disposal.

This sign should always be clear and clean, unobstructed and visible.

It is forbidden to dispose the device to containers of municipal waste.

Placed on the label.

2.6. Residual risk description and assessment

The device is designed and constructed in accordance with the current state of the art and the applicable standards.

Despite the efforts of the manufacturer to ensure the safety and elimination of hazards when using the SiDLY Care, some elements of risk during use cannot be avoided.

They usually result from the improper method of usage. The residual risk may arise in emergency situations, arising in particular from disregarding with the operating instructions or from not paying proper attention during user-to-device interaction.

The highest hazard occurs when conducting the following, prohibited activities described in section. 2.1 and 2.2 and:

- using the device in a manner not as intended purpose.
- using the device by children under three years old,
- exposing the unit to extreme temperatures.

It is possible to limit the residual risk to minimum provided that the following recommendations are fulfilled:

- carefully reading and following the operating instructions,
- control the level of battery charge.

3. SiDLY One DESCRIPTION

3.1. Intended use

Fig. 1. SiDLY One – overview

The device belongs to medical products of I class. It is the active non-invasive device used to generate alerts after pressing the SOS button or detecting the user's fall

The device should be used by people familiar with its intended use, and its technical characteristics.

SiDLY One is dedicated to cooperation with the www.sidly-platform.com system where it should be configured. The device is delivered to the user together with the installed SIM card, which allows the correct, provided that it works in a place where is a 2G GSM range.

The device is reusable until its functionality will be maintained. Is adapted to:

- send alarm information when pressing the SOS button on the device or detecting a fall,

in addition to:

- to designate geolocation coordinates (only if there is a GSM range

- possible to determinate geolocation and the GPS signal),
- signal the need for taking medications
- send alarm information onto the declared phone number,
- detect the user's falls if he or she stays down for more than 120 seconds after falling.

Unauthorised changes to the device without the manufacturer permission exempt him from liability for damages.

Meeting the requirements regarding the use of the device in accordance with the manufacturer's instructions and strict compliance with the recommendations is a condition of intended use. Using the device for other purposes will be considered as improper use.

In particular, it is unacceptable to use of the device by children under 3 years old.

3.2. Technical characteristics and construction

Table 1

Parameter	Unit	SiDLY Care
Weight	g	45
Overall dimensions:		
width	mm	max 44
height	mm	max 224
depth	mm	max 15
adjustment range (on the wrist)	mm	173 - 248
Minimum GSM signal strength required for communication	dBm	-108
The range of signal strength for proper operation	dBm	-95
The pulse measurement range	bpm	0 - 250
Battery charging time	h	2 - 3
Temperature of the device storage and use	°C	od -5 do +45
Life cycle	years	2
Water resistance	IP	65
Charging voltage	V	5
Maximum charging current	mA	1000
Charging socket	-	mikro USB

Table 2

Parameter	Unit	Charger for SiDLY One
Charger	type	GSM0811
mains supply	V (AC) / Hz	100-240 / 50-60
output voltage	V (DC)	5
device protection class	-	II

The device design is made of medical elastomer and polypropylene (which has been examined in terms of allergies, irritation and cytotoxicity).

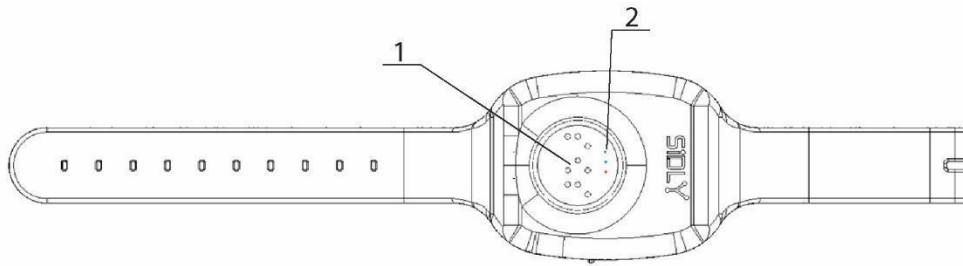
The device has a form of wristband with adjustable belt fastening on the wrist.

The SOS button described also in Braille is placed at the front shield. The 3 LED warning are placed on the button, each in a different color.

There is a protective cap of micro USB connector on the side and switch operated with a special tool (supplied with the unit) (fig. 3).

The device label is placed on the back of the cover (fig. 4).


The device is also equipped with an internal speaker and a microphone that allows to make voice calls.

1 – SOS button labeled in Braille

2 – LED warning

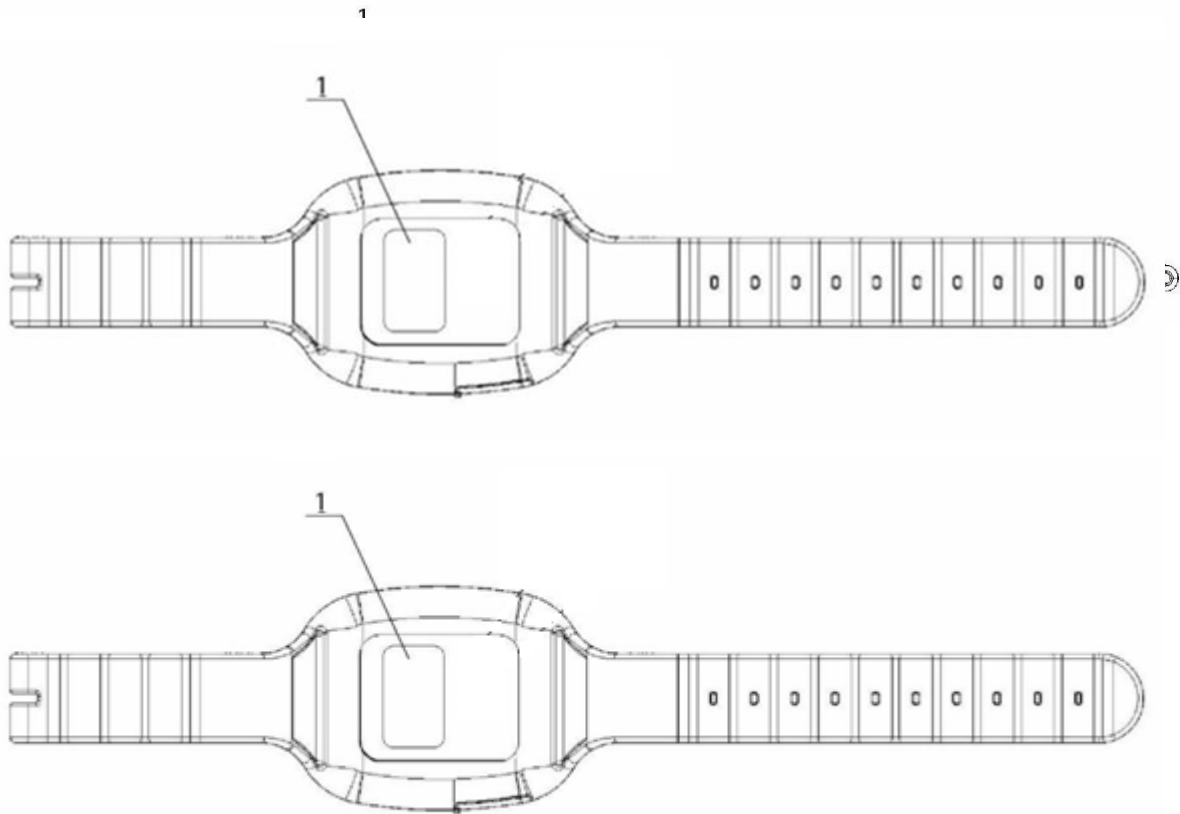Fig. 2. SiDLY One – rear view

1 – device label

Fig. 4. SiDLY One – rear view

4. PRELIMINARY OPERATIONS AND PUTTING INTO SERVICE

4.1. Device delivery

The device is supplied as a ready-to-use product. It is packaged in a cardboard box, which in addition to the devices are:

- dedicated charger,

- operating instructions,
- warranty

Read these instructions before you use the device.

5. USE OF DEVICE

5.1. General principles

Before each use, check the technical condition of the device. In case of deformation, cracks or other mechanical damage such a device should be withdrawn from use.

SiDLY One is designed for wearing on the wrist (it is recommended to place it

on the left wrist) in a manner such as on the figure below (fig. 5).

It should be placed on the wrist after wearing on clothes. Putting on clothes in a situation when the band is placed on wrist may damage the device - clothes may catch device, damaging them.

Fig. 5. SiDLY Care – placed on left wrist

5.2. Initial start

In the device is pre-installed SIM-card, which ensures the correct operation of the device, provided that the device works in a place where is a 2G GSM range.

During the initial activation procedure You must connect the device to the charger at first and carry out battery formation. It's done by fully device

charging. An estimated time to charge your device is 2 hours. Full charge will be signaled by red diode going off.

Initial start may take up to few minutes, which is due to the fact that SiDLY One has to download the configuration from the system.

5.3. Switching On and Off

Turning on the device is done by connecting SiDLY One to charger, which is connected to the power supply.

To start or turn off device:

- Raise the cap placed on the side of the device,
- Insert the micro USB plug of the charger into the device,
- The correct start of the the device will be signaled by a voice message: “Device is running”.

Turning off the device takes place after full discharging of installed in the device.

There is no other option to turn off the device.

After the device is correctly turned on, the device will be signaled on using the following successively:

- longer sound signal,
- continuous green LEDs (up to 10s),
- two short beeps (up to 30 seconds from connecting the device to the charger).

IMPORTANT!

The device is started by connecting to the charger. In case the battery installed in the device may not start immediately after connecting the charger. It is necessary to disconnect the charger after 10 minutes and reconnect it.

5.4. Device synchronization

In order to synchronize the device with an application should be made the user registration on www.sidly-platform.com or other website dedicated by the manufacturer. During the registration process, follow the instructions.

After a successful installation, add a band to the application..

To do this, enter the username, the IMEI number of the device which is available on the label of the product placed on the bottom of the device or on the label placed on the packaging and at least one emergency call.

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Notatka

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Wróć Dalej

Fig. 6. Application menu for SiDLY One used to add a new user.

WARNING!

It is required to enter at least one emergency telephone number.
Entering of other two is an option.

WARNING!

Only the SMS messages informing about the fall and pressing the SOS button are sent to the guardian's phone number and the voice call is made to first emergency number does not answer the call, the device will automatically call the second number in the order in which the emergency number will be announced, when the situation repeats for the second number, calls will be made to the third number. If none of the three numbers answers the call, the system will show a message that the connection can not be made. Notifications about the battery status and information on the results of measurements are sent to the system.

WARNING!

You have to enter the IMEI number correctly. To change the IMEI number You should delete the band from application and add it again.

After a maximum of 30 minutes from carrying the above steps, you will receive a message SMS on emergency phone number entered in application. A message is as follows
„Congratulations! Your SiDLY One has been configured and is ready for use!”

The above procedure occurs only when the device is switched on for the first time.

If you change the configuration settings, the new data will be updated in the device within 6 hours.

5.5. The device functionality

There are active the device functionality when you first switch the device on:

- make a voice call,
- medicines reminder,
- SOS function,
- GSM / GPRS Communication,
- fall detector.

There is required a cooperation with a dedicated platform www.sidly-platform.com for proper operation of all device functionality.

The following figure shows information about the LED colours and their reaction depending on the functions carried out by the device.

MINI GUIDE OF YOUR SiDLY CARE USAGE

-  proper device operation
-  low battery
-  parameters measurement (finished by 3 beeps)
-  information about exceeding the declared measurement values (after parameters measurement)
-  pressing the SOS button (the procedure is finished by 10 signals SOS in Morse code)
-  medicines reminder (supported by sound signals)
-  the fall signalling (after previous 10 short beeps)

LEGEND

-  continuous LED
-  flashing LED

Fig. 7. Mini guide of Your SiDLY Care usage

5.5.1. Determining a user's geolocation

Using A-GPS the device is able to locate the user's band.

Locating the user is performed when you press the SOS button and when it detects a falling.

The location of the user is transmitted by a link to openstreetmap.org or Google Maps in an SMS message sent on the declared emergency number, and presented on website www.sidly-platform.com.

Location accuracy resulting from used technology is up to 500 m. The manufacturer is not liable for exceeding the discrepancies localization. Location discrepancy may be result of insufficient level of telecommunication infrastructure expanding of mobile network operator.

In the case of lack of the band connection to the GPRS network location determining may not occur.

5.5.2. Medicines reminder

The device signals time for taking medicines declared by the user. User declares hours of taking medicines in the mobile application.

Along with the time declared enter the name of medicine or medicines that you need to take at certain times.

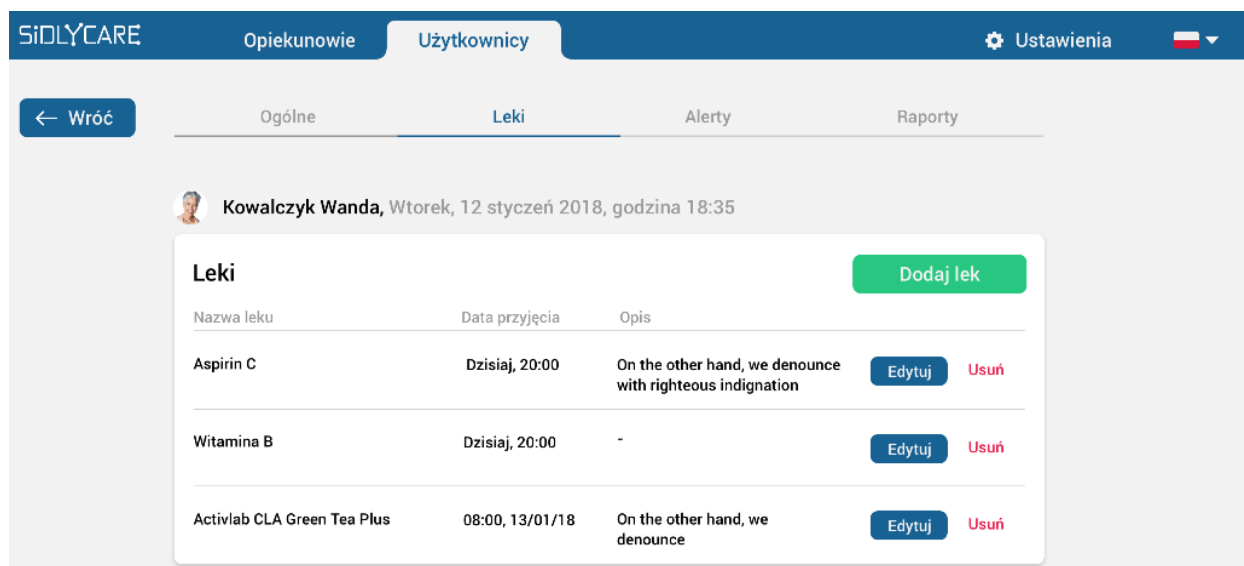
Fig. 8. Menu of www.sidly-platform.com – medicines reminder.

The device signals time for taking medicines using beeps and at the same time light signals - flashing blue LED.

The user can declare up to 12 different alarm times to remind of medicines.

5.5.3. SOS button / Panic Button

Calling SOS Alarm is done by holding down the SOS button. Start of the alarm procedure signals red LED. Then the band gets the location data and after about 1 - 2 minutes sends an SMS to the declared numbers ICE (In Case of Emergency) containing header SOS !!! and the location of the band. [SOS!!! (and link redirecting to the map with the location of the band)]. After it send an SMS successful there is sent to the www.sidly-platform.com. Then, a voice call is established to the first emergency number declared in the system. Then, a voice call is established to the first emergency number declared in the system. In case when the user of the first emergency number does not answer the call, the device will automatically call the second number declared as emergency number, when the situation repeats for the second number, calls will be made to the third number.

If none of the three numbers answers the call, the system will show a message that the connection can not be made.

WARNING! For the properly working of this function, you need the telephone network coverage for the band worn by the user, as well as for the telephone of person, to which the notifications are sent, because it has been declared as an emergency number (ICE number).

5.5.4. Fall detector

The SiDLY has a fall detector built-in the device.

During the first configuration, the fall detector sensitivity should be sent in the system www.sidly-platform.com.

The fall detector can take one of 6 levels, from 0 – detector off, to 5 – very sensitive detector. Setting the fall detector on level 5 can detect false fall alarms.

In case of suspected detection of fall SiDLY One generate a voice message: “Warning – possible downfall detected”. For the moment of generating the above message for the next 10 seconds the user of the can cancel the automatic help call.

False detection of the fall, user can cancel by holding the SOS button. If the alarm is not canceled, a continuous red LED will remain on until the end of the whole procedure.

In the case of a fall detection, an SMS is being sent with the text “Warning!!! – Possible fall” and sending message to the internet platform www.sidly-platform.com.

After sending message to the internet platform, a voice connection will be established to the first emergency number declared in the system. In case when the user of the first emergency number does not answer the call, the device will automatically call the second number declared as emergency number, when the situation repeats for the second number, calls will be made to the third number. If none of the three numbers answers the call, the system will show a message that the connection can not be made.

WARNING! For the properly working of this function, you need the telephone network coverage for the band worn by the user, as well as for the telephone of person, to which the notifications are sent, because it has been declared as an emergency number (ICE number).

WARNING!

SOS alarm activation and fall detection by the device results in sending SMS messages to declared numbers ICE and to the platform www.sidly-platform.com

5.5.5. GPRS communication

The device has the ability to connect to the Internet, making it possible to transfer measurement data from a band to the system and the ability to configure the band settings. For this purpose, a SIM card is being installed in the device.

The configuration between the band and the system is carried out via www.sidly-platform.com. To configure the SiDLY One device, the serial number (IMEI) of the device must be assigned to the user data. IMEI number is available on the product label, located at the bottom of the device or one the label on the box.

WARNING!

All settings made in the band and changes of these settings will be updated during the band connection with GPRS.

5.6. Battery charging

SiDLY One has a built-in lithium-polymer battery and charger dedicated for this device (fig. 16).

Charger parameters for SiDLY:
Travel Charger,
Model: GSM0811,
Input: AC 100 - 240 V / 50 – 60 Hz,
Output: DC 5 V, 1000 mA.

Fig. 9. Certified charger dedicated for the device

NOTE!

It is forbidden to charge your SiDLY when the user has it on his hand.

Battery charging

Connect SiDLY One to a power source using the supplied micro-USB charger.

Rechargeable batteries have a limited number of charge cycles. Frequent charging will shorten battery life.

To charge SiDLY One:

- remove the band from the user's hand,
- put the electric plug into the socket.
- place the charger the micro-USB plug in micro-USB slot of band,
- check that the LEDs indicating the charging lit up (red and green).

To complete the charging of SiDLY Care:

- check whether the red LED doesn't lights up, this is the way to indicate the end of charge,

- unplug the micro-USB plug from micro-USB slot of band,
- unplug charger from the electricity outlet.
- check whether the green LED flashes periodically to indicate normal operation.

To complete the charging of SiDLY One:

- check whether the red LED doesn't lights up, this is the way to indicate the end of charge,
- unplug the micro-USB plug from micro-USB slot of band,
- unplug charger from the electricity outlet.
- check whether the green LED flashes periodically to indicate normal operation.

Interruption of device charging by unplugging it from the power will cause that the red and green LEDs go off and then the green LED will flash periodically in case of proper further work.

It is required to fully charge the battery when you switch the device a first time. The user is informed about the charging device a message sent to the mobile applications and SiDLY One online platform reads: "The band [user] fully charged."

The band user is informed about the necessity of charging the band when the red LED diode starts blinking (if the band is not connected to the charger). In the case of close low battery message is sent to the Care SiDLY mobile applications and web-based platform

reads: "The band [User] discharged. Connect the charger to the band. "

How to use the charger can contribute to cable cracking and breaking. Charger cable may be weakened or become breakable if it is repeatedly bent in the same place. Bend cable smoothly, without breaking it. We recommend you to check frequently the cable and connectors condition, searching for holes, cracks and other damages. You should stop using the cable if there occurs described damages.

WARNING!

The device is recommended to charge until it sends a notification of readiness to use on the mobile applications and SiDLY Care online platform.

It is likely that the device switched off too early can quickly unload - it results from characteristics of lithium-polymer charging.

6. MAINTENANCE

6.1. Cleaning

If SiDLY One comes into contact with any substance likely to cause contamination (eg. dust, dirt, ink, cosmetics), the device should be immediately cleaned.

To clean SiDLY Care:

- to clean, use a soft, lint-free cloth or cleaning and disinfecting wipes,
- do not let moisture get into the inside of the device,
- do not use cleaning agents (detergents) or compressed air.

The device does not require special maintenance.

Long lasting and efficient use of the device is dependent on the skillful handling and correctness of cleaning soiled device.

Daily cleaning service is based on cleaning the device by cleaning and disinfecting tissue.

6.2. Storage

We recommend non taking the device off for night. This will prevent the patients that they forget about again its taking on.

If, however, you have to take the device off, put it in a safe place without exposure to fall from a height or losing.

We recommend charging the device at fixed times to not forget about the charging and reusing.

It is unacceptable to charging or store the device in an ambient temperature above 60 °C (risk of explosion).

7. WITHDRAWAL

WARNING!

Do not dispose the device in municipal waste. It is subject to the same rules as disposal of electronic products.

Guaranteed life cycle of the device is 12 months (see warranty conditions). This does not mean, however, that af-ter 12 months the device is totally in-operable. With careful handling of the

device it is possible that it works longer than 12 months.

In the case of the total device consumption or damage it must be removed from service and sent for disposal.

8. TESTS AND CERTIFICATES

WARNING!

Unauthorized changes or modifications by SiDLY may violate electromagnetic compatibility (EMC) and cause the loss of the right to claim warranty. The product complies with EMC requirements, among others, in terms of cooperation with compatible devices (charger).

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
The device has been tested for compliance with the requirements of Council Directive 93/42/EEC by TÜV Rheinland (Notified Body No 0197) and obtained CE certificate.

9. WARRANTY

SIDLY Sp. z o.o.
2/31 St. Chmielna
00-020 Warsaw, PL
NIP:7010435677
Regon:147363415
KRS:0000519291

Stamp of seller

WARRANTY

SIDLY One device

Serial number

Year of manufacture.....

designed to monitor important life parameters.

Date of sale.....

Buyer.....

Period of warranty validity is 12 months from date of delivery.

Warranty service is performed by the producer.

.....
Date

.....
Signature of the seller

When complaints you must show warranty card and proof of purchase (invoice, receipt).

The buyer receives the warranty from the supplier for a period of 12 months counting from the date of device delivery.

NOTE FOR THE BUYER!

The purchaser should carefully read a warranty card.

GENERAL RULES OF WARRANTY PROCEEDINGS

- 1) The warranty does not exclude, not limit or not suspend the rights of the buyer arising from the provisions of the statutory warranty for defects in the goods sold.
- 2) Warranty covers defects and damage of the device caused by the fault of the producer, due to material defects, improper producing or assembly. By provision of guarantees the manufacturer undertakes to complete device replacement for a new device free of defects.
- 3) The Guarantor declares that subject covered by this Warranty has been issued free of defects and was manufactured in accordance with current standards.
- 4) The user notifies complaint to a dealer from which the machine was purchased or to the manufacturer (address on the warranty card).
- 5) The manufacturer will consider the complaint as soon as possible and no later than within 21 calendar days from the date of delivery defective product by the consumer to the guarantor.
- 6) The user should submit a complaint at the latest within 14 days after the date of damage.
- 7) The warranty is extended by a period in which a complaint was considered, counting from a date of filing the complaint and until the day of providing a new device free of defects to the user.
- 8) The manufacturer will not accept warranty claims if there were made any technical changes in a device without the manufacturer approval, if device was improperly stored, operated or maintained. It also concerns identified prior at-tempts to repair the device.
- 9) The warranty does not cover:
 - a) defects caused by the machine operation not in accordance with the principles of operation, by alterations, design changes made by the user or third parties;
 - b) defects caused by user abandon of activities related to the machine maintenance, cleaning and adjustment;
 - c) defects caused by external influences and particularly mechanical damage, damage due to the action of chemicals;
 - d) natural wear and tear consistent with the properties or intended device use including;
 - e) the damages caused by mechanical, thermal, chemical, and other factors caused by user activity or by external forces;

- f) damages that occurred as a result of transport and during loading machines operated by any carrier.
- 10) The producer is not obliged to bear the cost of compensation due to production losses or any other damages resulting from damage, hidden defects or device failure.
- 11) This guarantee does not affect the rights and obligations arising from the provisions of national law.
- 12) The warranty is voided if there are made modifications, adaptations or changes in machinery construction by persons other than the manufacturer or the person designated by him.

ELEMENTS TO BE RETURNED TO THE MANUFACTURER.

1. Complete device with packaging, charger and proof of purchase. I

have read the warranty term and conditions

.....
Date and user signature